



## SMART RAPID SINGLE PATIENT

### ABSOLUTE HYGIENE

Developed in close cooperation with hygiene authorities and specially designed for one-time application, the single patient cuff is ideal for use as part of an integrated hygiene concept. The single use of the cuff significantly minimises risk of cross-contamination in everyday medical practice. The materials used are especially skin friendly, dermatologically safe and ensure comfortable wear for the patient. The clearly visible position indicators enable flawless application of the cuff - consisting of the cover and integrated inflation bladder - for optimal measurement results. Quality and functionality - here at ERKA. we constantly strive for perfection and apply this high standard to all of our products. To that end, our quality control is rigorous: each and every ERKA GREEN CUFF is tested for proper operation - this is our ERKA. 100% Quality Check. In addition, our cuffs fulfill all requirements of the latest AAMI and AHA guidelines for correct blood pressure cuff size.

### HOW CAN THE RISK OF INFECTION BE MINIMIZED FOR BOTH - THE PATIENT AND MEDICAL PROFESSIONALS?

The surest protection is the use of recyclable disposable cuffs. We offer a comprehensive range here. The intentional omission of PVC and DEHP in our products and optimal supply chain combine ecology and efficiency in the most impactful way to act responsibly.

### WHICH ALTERNATIVE PRODUCTS CAN I USE INSTEAD OF THE RECYCLABLE DISPOSABLE CUFF?

All reusable ERKA GREEN CUFF blood pressure cuffs can be disinfected thoroughly and quickly because of their unique surface and materiality.

### IS THERE EVEN MORE PROTECTION PROVIDED BY ERKA?

It is recommendable to use ERKA reusable cuffs with the matching ERKA hygiene protection stripes (Ref. 000.04100). This prevents a direct skin contact and minimizes therefore potential risks.

### WHY THE SELECTION OF THE RIGHT CUFF SIZE IS SO IMPORTANT?

So-called undercuffing or overcuffing i.e. the selection of a cuff which is too small or too large relative to the upper arm circumference can lead to a distortion of the measured values by up to 30 mmHg. Each ERKA cuff has clear positioning instructions and clearly recognizable size labels.

### HOW CAN I CHANGE THE CUFF SIZE?

Cuff change in the blink of an eye - all ERKA SMART line blood pressure devices and cuffs are equipped with a quick closure so cuffs can be changed in a second. Quickly, conveniently and error-free for optimal values.

### WHICH BLOOD PRESSURE MEASUREMENT INSTRUMENTS ARE PART OF THE SMART LINE?

ERKAMETER EERKAMETER E flexERKAMETER 125 PROKobold 48 SMARTSWITCH 2.0 SMART

### SMART

Smart: with the ERKA quick closure, cuffs can be changed in seconds - the right cuff for every patient.

### RAPID FASTENER

Rapid - clever like a velcro: the super soft adhesive and fleece strap on the cuff closes this simple and reliable in every application.

### SINGLE PATIENT

Single patient means a holistic health concept for patient and medical staff.

### HOW CAN THE REUSABLE ERKA CUFF BE CLEANED

The following ERKA GREEN CUFFS are equipped with a water resistant, easy to clean surface and therefore can be disinfected easily and thoroughly: Superb Rapid Superb rapid one piece Superb Hook Superb D-Ring Superb Home

### HOW CAN THE REUSABLE ERKA CUFF BE DISINFECTED?

All ERKA reusable cuffs may be disinfected thoroughly by spraying or wiping with tested approved disinfectants. Afterwards the surface is wiped with light pressure.

### WHICH DISINFECTANTS ARE RECOMMENDED BY ERKA?

Fugaten® Spray Esemfix® from Schülke Promanum® pur from B.Braun Meliseptol®  
Foam pur from B.Braun

#### WHY ALL ERKA GREEN BLOOD PRESSURE CUFFS ARE SO PLEASANT TO USE?

The optimal comfort is determined by the combination of materials used and the complete absence of PVC , DEHP and similar substances. Furthermore, our materials are biocompatible and satisfy the requirements specified in DIN ISO 10993.

#### HOW TO PLACE THE ERKA CUFF CORRECTLY?

To avoid any error the cuff must be placed correctly. The brachial artery is clearly marked on all ERKA cuffs: it is a reliable indicator for positioning the cuff.

#### HOW TO CHOOSE THE CORRECT SIZE OF THE CUFF?

All ERKA GREEN CUFFs are clearly marked with a size range. One must choose a different cuff size if the arm circumference does not fit into one of the ranges.

Smart Rapid single patientSize	Reference
Box of 10 - Size 1024.50473.	
Box of 10 - Size 5024.54473.	
Box of 10 - Size 1024.56473.	
Box of 10 - Size 2024.57473.	
Box of 10 - Size 1024.58473.	

Accessories

Size

Reference

Spare Parts

## Legal Notes and Data Protection

ERKA.  
Kallmeyer Medizintechnik GmbH & Co. KG  
Im Farchet 15  
83646 Bad Tölz

Geschäftsführer: Björn Kallmeyer

HRA 75980, Amtsgericht München  
Umsatzsteuer-ID: DE 812 833 893

Kontakt:  
Telefon: 0 80 41 / 80 09-0  
Telefax: 0 80 41 / 80 09-39  
eMail: [made@erka.org](mailto:made@erka.org)  
<http://www.erka.org>

Verantwortlich:  
Björn Kallmeyer

## Signatur / Stempel

HOW TO CHOOSE THE CORRECT SIZE OF THE CUFF?

