



## SUPERB RAPID

### YOUR CHOICE FOR EVERYDAY USE

In modern clinical work, the issue of hygiene is ubiquitous and of paramount importance - for the benefit of patients and medical staff. Precisely for this reason, all ERKA. GREEN CUFF Superb cuffs are biocompatible, easy to disinfect and equipped with a waterproof surface that can be thoroughly cleaned. The ERKA. GREEN CUFF Superb cuffs consist of a cover and a removable inflation bladder. This construction is optimal in blood pressure measurement because of even pressure distribution and the resulting highly precise values. Of course, the ERKA. GREEN CUFF Superb fulfils all requirements of the latest AAMI and AHA guidelines for proper blood pressure cuff size. In addition, clearly indicated size and positioning instructions ensure a proper selection and flawless application of the cuff.

### HOW CAN THE RISK OF INFECTION BE MINIMIZED FOR BOTH - THE PATIENT AND MEDICAL PROFESSIONALS?

The surest protection is the use of recyclable disposable cuffs. We offer a comprehensive range here. The intentional omission of PVC and DEHP in our products and optimal supply chain combine ecology and efficiency in the most impactful way to act responsibly.

### WHICH ALTERNATIVE PRODUCTS CAN I USE INSTEAD OF THE RECYCLABLE DISPOSABLE CUFF?

All reusable ERKA GREEN CUFF blood pressure cuffs can be disinfected thoroughly and quickly because of their unique surface and materiality.

### IS THERE EVEN MORE PROTECTION PROVIDED BY ERKA?

It is recommendable to use ERKA reusable cuffs with the matching ERKA hygiene protection stripes (Ref. 000.04100). This prevents a direct skin contact and minimizes therefore potential risks.

### WHICH ERKA BLOOD PRESSURE MEASURING INSTRUMENTS ARE SUITED FOR SINGLE TUBE CUFFS?

Kobold Switch 2.0

### DUE TO THE UNIQUE MEMBRANE-PRESERVING TWIN TUBE TECHNOLOGY THE FOLLOWING DEVICES ARE EQUIPPED WITH TWIN TUBE CUFFS:

Perfect Aneroid 48 Perfect Aneroid 56 Perfect Aneroid Klinik 48 Perfect Aneroid Klinik 56 Profi 48 Profi 56 Profi Klinik 48

### FOR A MAXIMUM OF OPERATING CONVENIENCE ALL FIXED DEVICES ARE EQUIPPED WITH DOUBLE-TUBE CUFFS.

Erkameter 3000 Erkameter 3000 eco Vario

### **SUPERB**

Superb stands for the uncompromising design free of PVC, DEHP and latex. Both - in the choice of materials and in the manufacturing process.

### **RAPID FASTENER**

Rapid - clever like a velcro: the super soft adhesive and fleece strap on the cuff closes this simple and reliable in every application.

### **HOW CAN THE REUSABLE ERKA CUFF BE CLEANED?**

The following ERKA GREEN CUFFS are equipped with a water resistant, easy to clean surface and therefore can be disinfected easily and thoroughly: Superb Rapid Superb rapid one piece Superb Hook Superb D-Ring Superb Home Smart rapid Smart rapid one piece

### **HOW CAN THE REUSABLE ERKA CUFF BE DISINFECTED?**

All ERKA reusable cuffs may be disinfected thoroughly by spraying or wiping with tested approved disinfectants. Afterwards the surface is wiped with light pressure.

### **WHICH DISINFECTANTS ARE RECOMMENDED BY ERKA?**

Fugaten® Spray Esemfix® from Schülke Promanum® pur from B.Braun Meliseptol® Foam pur from B.Braun

#### WHY ALL ERKA GREEN BLOOD PRESSURE CUFFS ARE SO PLEASANT TO USE?

The optimal comfort is determined by the combination of materials used and the complete absence of PVC , DEHP and similar substances. Furthermore, our materials are biocompatible and satisfy the requirements specified in DIN ISO 10993.

#### HOW TO PLACE THE ERKA CUFF CORRECTLY?

To avoid any error the cuff must be placed correctly. The brachial artery is clearly marked on all ERKA cuffs: it is a reliable indicator for positioning the cuff.

#### HOW TO CHOOSE THE CORRECT SIZE OF THE CUFF?

All ERKA GREEN CUFFs are clearly marked with a size range. One must choose a different cuff size if the arm circumference does not fit into one of the ranges.

Superb RapidSize	Reference
Size 4	025.20481.
Size 4	025.20482.
Size 4	025.20491.
Size 4	025.20492.
Size 4	025.20493.
Size 5	025.24481.
Size 5	025.24482.
Size 5	025.24491.
Size 5	025.24492.
Size 5	025.24493.
Size 3	025.26481.
Size 3	025.26482.
Size 3	025.26491.
Size 3	025.26492.
Size 3	025.26493.
Size 2	025.27481.
Size 2	025.27482.
Size 2	025.27491.
Size 2	025.27492.
Size 2	025.27493.

# ERKA.

THE ORIGINAL

Made in Germany since 1889

Size 1 025.28481.

Size 1 025.28482.

Size 1 025.28491.

Size 1 025.28492.

Size 1 025.28493.

Size 6 025.29481.

Size 6 025.29482.

Size 6 025.29491.

Size 6 025.29492.

Size 6 025.29493.

Accessories

Size

Reference

Spare Parts



Legal Notes and Data Protection

ERKA.  
Kallmeyer Medizintechnik GmbH & Co. KG  
Im Farchet 15  
83646 Bad Tölz

Geschäftsführer: Björn Kallmeyer

HRA 75980, Amtsgericht München  
Umsatzsteuer-ID: DE 812 833 893

Kontakt:  
Telefon: 0 80 41 / 80 09-0  
Telefax: 0 80 41 / 80 09-39  
eMail: [made@erka.org](mailto:made@erka.org)  
<http://www.erka.org>

Verantwortlich:  
Björn Kallmeyer

Signatur / Stempel

